
DIRECTIONS FOR USE

ADULT AND PEDIATRIC SINGLE-PATIENT USE SENSOR

To Apply the Sensor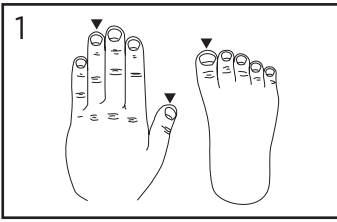


1. Select the sensor site on the patient. For adults, the fourth finger is recommended, as this keeps the sensor out of the patient's way. For pediatrics <30 kg, the thumb or big-toe is recommended.


2. Remove the sensor from its package, and inspect for damage. Return damaged sensors immediately for warranty-replacement.

3. Remove the release liner from the sensor. Align the Detector (☺) underneath the finger or toe.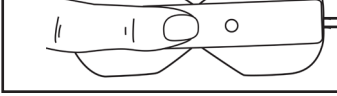

4. Wrap adhesive-tape flaps on the detector. Fold the adhesive-tape flaps cover the nail, reposition to a larger finger or toe.

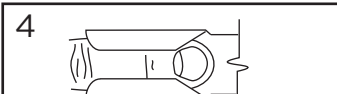5. Align the Emitter (☼) directly opposite the Detector (☺), on the top of the nail bed.

6. Press the Emitter (☼) of the adhesive-tape down on the top of the finger. Fold the adhesive-tape flaps down around the sides of the finger. Pinch any openings in the tape shut. Do not pull or stretch the tape during application.


7. This sensor may be connected directly to a CSI Oximeter. You may extend the length by using the Model 4441 (10 ft) CSI Patient Monitor Extension Cable.

INFANT AND NEONATAL SINGLE-PATIENT USE SENSOR

To Apply the Sensor

1. Select the sensor site on the patient. For infants (10 to 20 kg), the side-of-foot or thumb is recommended. For neonates (<10 kg), the side-of-foot or side-of-hand is recommended.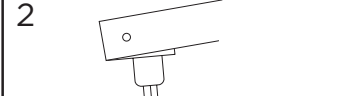


2. Remove the sensor from its package, and inspect for damage. Return damaged sensors immediately for warranty-replacement.

3. For side-of-foot placement: position Emitter (☼) underneath foot, proximal to 5th toe. Wrap adhesive-tape around top of foot. Detector (☺) should be directly opposite the Emitter (☼).

4. For side-of-hand placement: position Emitter (☼) underneath hand, proximal to 5th finger. Wrap adhesive-tape around hand. Detector (☺) should be directly opposite the Emitter (☼).

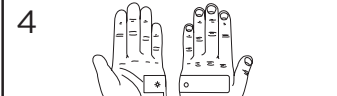5. For great-toe or thumb placement: position Emitter (☼) underneath toe or thumb. Wrap adhesive-tape around site of finger or toe. Detector (☺) should be on top of nail-bed directly opposite the Emitter (☼).

6. This sensor may be connected directly to a CSI Oximeter. You may extend the length by using the Model 4441 (10 ft) CSI Patient Monitor Extension Cable.

7. This sensor may be connected directly to a CSI Oximeter. You may extend the length by using the Model 4441 (10 ft) CSI Patient Monitor Extension Cable.

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