**SpO<sub>2</sub>**  
Finger Sensor  
Reusable, Adult

#### DIRECTIONS FOR USE

#### CSI-Model

##### Intended Use

OSI Optoelectronics Inc. BluSat™ oximetry sensors are indicated for use in continuous noninvasive monitoring of arterial oxygen saturation and pulse rate. This BluSat sensor is compatible with CSI Oximeters.

##### Precautions


- For further instructions, warnings and precautions, refer to the operating instructions provided with your monitor.
- Do not autoclave or immerse the sensor in liquid.
- CAUTION: Federal (USA) law restricts this device to be sold by or on the order of a physician.

##### Warnings

- This sensor is not to be used on any other monitor other than the above specified brand of oximeter monitor. Using this sensor on oximeters not specified may cause patient injury, inaccurate measurements or equipment damage.
- Check and reposition the sensor every 4 to 8 hours and reposition to an alternate site every 24 hours.

##### To apply the sensor

- Connect the sensor cable to the OSI Optoelectronics Inc. adapter cable. Turn on the oximeter and verify proper operation.
- Insert the patient's digit into the sensor. The preferred digit to use for adults is the index finger. The digit is correctly inserted when:
  - The tip of the digit touches the rear guide posts.
  - The sensor cable extends along the top of the patient's hand.
- Visually monitor the sensor site to ensure the integrity of the skin.

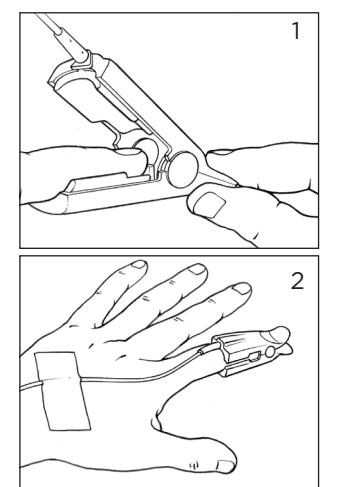##### To Clean the Sensor

To clean the sensor between uses or between patients, wipe the sensor surfaces with a soft cloth moistened in water, mild soap solution, or isopropyl alcohol.

##### Environmental Requirements

- Operating temperature range: 50 - 104° F (10 - 40° C)
- Storage and transportation temperature range: 32 - 122° F (0 - 50° C)
- Relative Humidity: 0 - 75% non-condensing

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- Avoid application of the sensor to edematous or fragile tissue.
- Excessive patient motion or ambient light, electromagnetic interference, dysfunctional hemoglobin, low perfusion, intravascular dyes, finger nail polish and long or artificial finger nails may affect the sensor performance and the accuracy of the measurement.
- CAUTION: Federal (USA) law restricts this device to be sold by or on the order of a physician.
- Do not use the sensor if it is damaged. Use of damaged sensor could cause patient injury or equipment failure.
- The sensor is contraindicated for use on patients who exhibit allergic reactions to adhesive tape.
- Note: The materials used in the manufacture of the sensor contain no natural latex protein. The materials have undergone extensive biocompatibility testing. Further information is available upon request.

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herein are trademarks of their respective